651 PROCONNECT
BLOOD PRESSURE MONITOR
UA-651BLE

Instruction Manual
ORIGINAL

1WMPD4003261

Greetings

This A&D blood pressure monitor is one of the most advanced monitors available and is designed for ease of use and accuracy. This device will facilitate your daily blood pressure regimen. We recommend that you read through this manual carefully before using the device for the first time. The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by A&D is under license. Other trademarks and trade names are those of their respective owners.

Preliminary Remarks

- ❑ The device complies with part 15 of the FCC rules and contains the FCC ID VPYLBVZ.
- ❑ The device is designed for use on adults only, and is not intended for infants.
- ❑ This device is designed for use in the home healthcare environment to measure blood pressure and pulse rate of people for diagnosis.

Precautions

- ❑ Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- ❑ Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
- ❑ Avoid tightly folding the cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components.
- ❑ The device and cuff are not water resistant. Prevent rain, sweat and water from soiling the device and cuff.

Precautions

- ❑ Measurements may be distorted if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong electrical fields.
- ❑ When using the device, confirm that the device is clean.
- ❑ Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.
- ❑ When the AC adapter is used, make sure that the AC adapter can be readily removed from the electrical outlet when necessary.
- ❑ Do not modify the device. It may cause accidents or damage to the device.
- ❑ To measure blood pressure, the arm must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the arm. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time.
- ❑ Take care to avoid accidental strangulation of babies or infants with the hose.
- ❑ Do not twist the air hose during measurement. This may cause injury due to continuous cuff pressure.
- ❑ Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this blood pressure monitor. Therefore, a minimum distance of 11 feet (3.3 meters) should be kept from such devices.
- ❑ Measuring blood pressure too frequently may cause harm due to blood flow interference. Check that the operation of the device does not result in prolonged impairment of blood circulation, when using the device repeatedly.

Precautions

- ❑ If you have had a mastectomy, please consult a doctor before using the device.
- ❑ Do not let children use the device by themselves and do not use the device in a place within the reach of infants.
- ❑ There are small parts that may cause a choking hazard if swallowed by mistake by infants.
- ❑ Do not apply the cuff on an arm in which another medical device is attached. The equipment may not function properly.
- ❑ People who have a severe circulatory deficit in the arm must consult a doctor before using the device, to avoid medical problems.
- ❑ Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatment.
- ❑ Do not apply the cuff on an arm with an unhealed wound.
- ❑ Do not apply the cuff on an arm receiving an intravenous drip or blood transfusion. It may cause injury.
- ❑ Do not use the device where flammable gases such as anesthetic gases are present. It may cause an explosion.
- ❑ Do not use the device in highly concentrated oxygen environments, such as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire or explosion.
- ❑ This device complies with Part 15 of FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operations.

Table of Contents

Parts Identification	E-6
Symbols	E-7 - E-8
Using the Monitor	E-9 - E-16
Measurement	E-17 - E-19

What is an Irregular Heartbeat	E-20
Pressure Bar Indicator	E-20

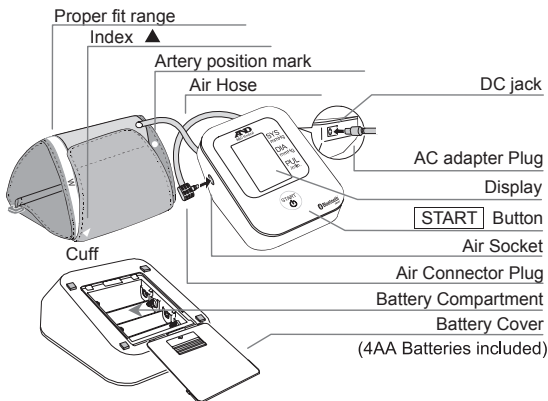
Troubleshooting	E-21
Maintenance	E-22
Technical Data	E-22 - E-25

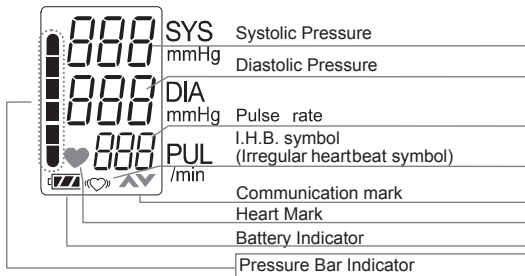
Contact Information	E-25
Warranty Information	E-26 - E-28

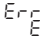
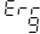
Parts Identification

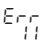
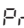
Display

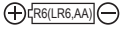
Symbols

Symbols	Function/Meaning	Recommended Action
	Appears while measurement is in progress. It blinks when the pulse is detected.	Measurement is in progress. Remain as still as possible.
	Irregular Heartbeat symbol (I.H.B.) Appears when an irregular heartbeat is detected. It may light when a very slight vibration like shivering or shaking is detected.	_____
	FULL BATTERY – The battery power indicator during measurement.	_____
	LOW BATTERY The battery is low when it blinks.	Replace all batteries with new ones, when the indicator blinks.
	Unstable blood pressure due to movement during the measurement.	Take another measurement. Remain still during the measurement.
	The systolic and diastolic values are within 10 mmHg of each other.	Apply the cuff correctly, and try the measurement again.
	The pressure value did not increase during inflation.	
	The cuff is not applied correctly.	
	Pulse display error. The pulse is not detected correctly.	
	Device internal error	Remove the batteries and press the ϕ button, and then install the batteries again. If the error still appears, contact the dealer.
		
	\wedge and \vee are illuminated alternately during communication	

Symbols

Symbols	Function/Meaning	Recommended Action
	Pairing has not been performed correctly	Remove and re-insert the batteries. Try pairing again.
		
	Pairing in progress	_____
	Unpairing in progress	_____
	Pairing complete	_____


Symbols printed on the device case

Symbols	Function/Meaning
	Standby and turn the device on.
SYS	Systolic blood pressure in mmHg
DIA	Diastolic blood pressure in mmHg
PUL/min	Pulse per minute
	Battery installation guide
	Direct current
	Type BF: Device, cuff and tubing are designed to provide special protection against electrical shocks.
IP	International protection symbol
SN	Serial number
	Refer to instruction manual/booklet
	Keep dry
	Polarity of DC jack
BT	Bluetooth Address

Using the Monitor

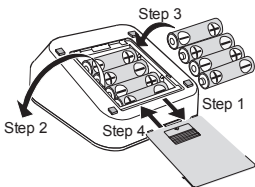Installing/Changing the batteries

1. Remove the battery cover.
2. Remove the used batteries from the battery compartment when changing them.
3. Insert new batteries into the battery compartment as shown, taking care that the polarities (+) and (-) are correct.
4. Replace the battery cover. Use only R6P, LR6 or AA batteries.

CAUTION

- Insert the batteries as shown in the battery compartment. If installed incorrectly, the device will not work.
- When  (blinks on the display and the device indicates that the battery needs to be replaced, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction. Replace the batteries two seconds or more after the device turns off.
- If  does not appear, then the batteries are completely drained.
- The battery life varies with the ambient temperature and may be shorter at low temperatures. Generally, four new LR6 batteries will last approximately for one year when used twice for measurement each day.
- Use the specified batteries only. The batteries provided with the device are for testing the device performance and may have a limited life.
- Remove the batteries if the device is not to be used for a long time. The batteries may leak and cause a malfunction.

Using the Monitor

Connecting the Air Hose

Insert the air connector plug into the air socket firmly.

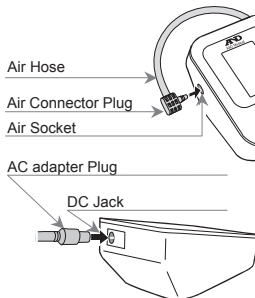
Connecting the AC Adapter

Insert the AC adapter plug into the DC jack. Then, insert the AC adapter into an electrical outlet. The AC adapter part # TB-233, is sold separately.

- Use the specified AC adapter. (Refer to E-22)
- When disconnecting the AC adapter from the electrical outlet, grasp and pull the AC adapter body out of the outlet.
- When disconnecting the AC adapter plug from the blood pressure monitor, grasp and pull the AC adapter plug out of the monitor.

Wireless Function

- In the unlikely event that this monitor causes radio wave interference to a different wireless station, change the location where this monitor is used or stop use immediately.
- Be sure to use in a location where visibility between the two devices that you want to connect is good. The connection distance is reduced by the structure of buildings or other obstructions. In particular, connection may be impossible when devices are used on either side of reinforced concrete.
- Do not use *Bluetooth*® connection in the range of a wireless LAN or other wireless devices, near devices that emit radio waves such as microwaves, in locations where there are many obstructions, or in other locations where signal strength is weak. Doing so may result in frequent loss of connection, very slow communication speeds and errors.

Using the Monitor

- ❑ Using close to an IEEE802.11g/b/n wireless LAN device may cause mutual interference to occur, which may result in reduced communication speeds or which may prevent connection. In this case, switch off the power supply to the device that is not being used, or use the monitor in a different location.
- ❑ If the monitor does not connect normally when used near a wireless station or broadcast station, use the monitor in a different location.
- ❑ A&D Medical cannot accept liability for any damages incurred due to impaired operation or data loss, etc. that occur through the use of this product.
- ❑ This product is not guaranteed to connect to all *Bluetooth*[®] compatible devices.

Bluetooth[®] Transmission

This product is equipped with a *Bluetooth*[®] wireless function and can connect to the following *Bluetooth*[®] devices.

Bluetooth[®] devices that can be connected:

- Continua certified devices
- iPhone, iPad, iPod (iPhone 4S or later)
- Applications and devices that are compatible with Bluetooth 4.0

Each device needs an application to receive data. For connection methods, refer to the manual for each device.

 **Bluetooth**[®] *Bluetooth*[®] Smart devices carry the *Bluetooth*[®] logo mark.

 **Continua**[®]
CERTIFIED Continua certified devices carry the Continua logo mark.

Using the Monitor

To connect with your mobile device - download and install the “A&D Connect” app:

Follow the instructions in the app to connect.
Check the compatible device list as the following website.
<http://www.wellnessconnected.jp/english/compatibility/>

Connecting to a mobile device (Pairing)

A *Bluetooth*® device needs to be paired with this blood pressure monitor in order to communicate with that device. If this monitor is paired with a receiver device from the start, measurement data is transmitted automatically to the receiver device each time a measurement is made.

Tips for pairing

- Be sure to turn the power of all other *Bluetooth*® devices off when pairing. Multiple devices cannot be paired at the same time. If the receiver device cannot receive measurement data, try pairing once again.
- Communication is possible with one of the receiver devices. If a second receiver device is paired, the device other than the most recently connected device will be unpaired to enable the new device to be paired.
- Follow the steps on the next page to pair the monitor with a *Bluetooth*® compatible receiver device. Also refer to the manual of the receiver device. Please use a pairing wizard if one is provided.

Using the Monitor

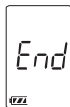
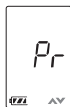
Pairing procedure

1. Follow the instructions in the manual of the receiver device to switch it to the pairable status. When pairing this monitor, place it as close as possible to the receiver device to be paired with.
2. Install the batteries or connect the AC adapter.

Press and hold the "START" button until "Pr" is displayed, then release the button.

The monitor will be in a state that can be found by the receiver device for about one minute.

3. The monitor displays "End" to indicate that pairing is complete.
4. If "Err 10" is displayed or pairing is failed, remove the batteries or disconnect the AC adapter and try steps 1-3 again.
5. Follow the manual of the pairing receiver device to search for, select and pair with this monitor.

Communication Distance

The communication distance between this monitor and the receiver device is about 33 feet (10 m). This distance is reduced by the conditions in the surrounding environment, so be sure to check that the distance is short enough for a connection to be made after measurement is complete.

Using the Monitor

Transmitting Temporarily Stored Data

In cases when the receiver device cannot receive measurement data, the measurement data is temporarily stored in the monitor memory. The data stored in the memory is transmitted the next time a connection is successfully made to the receiver device. A total of 30 sets of measurement data can be stored. When the amount of data exceeds 30 sets, the oldest data is deleted and the new data is stored.

Time

This monitor has a built-in clock. The date and time a measurement was taken is included in the measurement data. The built-in clock is designed to be automatically adjusted by syncing with the clock of a receiver device. Refer to the specifications of the receiver device. This monitor has no clock adjustment function.

Selecting the correct cuff

Using the correct cuff size is important for an accurate reading. If the cuff is not the proper size, the reading may yield an incorrect blood pressure value.

The arm size is printed on each cuff.

The index ▲ and proper fit range, on the cuff, tell you if you are applying the correct cuff. Refer to "Applying The Arm Cuff".

If the index ▲ points outside of the range, contact your local dealer to purchase a replacement cuff.

The arm cuff is a consumable. If it becomes worn, purchase a new one.

Using the Monitor


Arm size	Recommended Cuff Size	Part Number
8.6"-16.5" (22 to 42cm)	AccuFit™ Plus cuff	UA-420
9.0"-14.6" (23 to 37cm)	Medium SlimFit™	UA-290
12.2"-17.7" (31 to 45cm)	Large SlimFit™	UA-291

Arm size: The circumference at the biceps.

Note: Model UA-651BLE is not designed to use a small cuff.

Symbols that are printed on the cuff

Symbols	Function/Meaning	Recommended Action
●	Artery position mark	Set the ● mark on the artery of the upper arm or in line with the ring finger on the inside of the arm.
▲	Index	_____
REF	Catalog number	_____
M	Proper fit range for the medium cuff. It's printed on the cuff.	_____
L	Over range printed on the medium cuff and AccuFit Plus cuff.	Use the large cuff instead of the medium cuff or the AccuFit™ Plus cuff.
W	Proper fit range for the AccuFit™ Plus cuff. It's printed on AccuFit™ Plus cuff.	_____
L	Proper fit range for the large cuff. It's printed on the large cuff and AccuFit™ Plus cuff.	_____
S	Under range printed on the medium cuff and AccuFit™ Plus cuff.	_____
M	Under range printed on the large cuff.	Use the medium cuff instead of the large cuff.
LOT	Lot number	_____

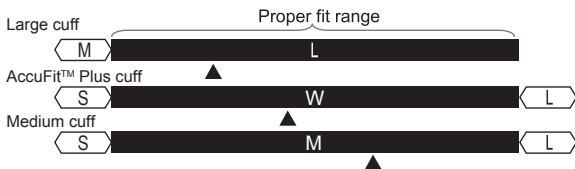Using the Monitor

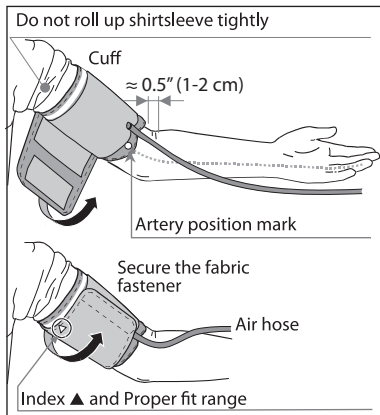
Applying the arm cuff

1. Wrap the cuff around the upper arm, about 0.5" (1-2 cm) above the inside of the elbow, as shown.

Place the cuff directly against the skin, as clothing may cause a faint pulse and result in a measurement error.

2. Constriction of the upper arm, caused by rolling up a shirtsleeve, may prevent accurate readings.
3. Confirm that the index ▲ points within the proper fit range.

Note: During measurement, it is normal for the cuff to feel very tight. (Do not be alarmed).

Measurement

Normal Measurement

1. Place the cuff on the arm (preferably the left arm). Sit quietly during measurement.
2. Press the **START** button. All of the display segments are displayed. Zero is displayed blinking briefly. Then the display changes, as indicated in the figure to the right, as the measurement begins. The cuff starts to inflate. It is normal for the cuff to feel very tight. A pressure bar indicator is displayed, as in the figure to the right, during inflation.

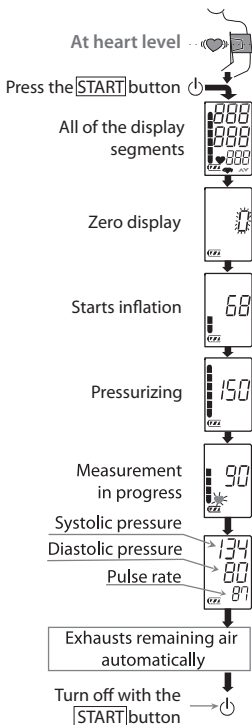
Note: If you wish to stop inflation at any time, press the **START** button again.

3. When inflation is complete, deflation starts automatically and the ♥ (heart mark) blinks, indicating that the measurement is in progress. Once the pulse is detected, the mark blinks with each pulse beat.

Note: If an appropriate pressure is not obtained, the device starts to inflate again automatically.

4. When the measurement is complete, the systolic and diastolic pressure readings and pulse rate are displayed. The cuff exhausts the remaining air and deflates completely.
5. Press the **START** button again to turn off the power.

Note: This product is provided with an automatic power shut-off function, which turns the power off approximately one minute after measurement.


Measurement

Measurement with the Desired Systolic Pressure

If re-inflation occurs repeatedly, use the following methods. If your systolic pressure is expected to exceed 230 mmHg, use this procedure.

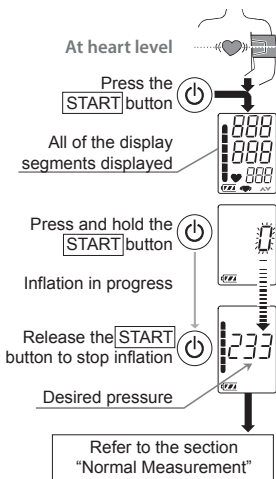1. Place the cuff on the arm at heart level (preferably the left arm).
2. Press the **START** button.
3. During the zero blinks, press and hold the **START** button until a number about 30 to 40 mmHg higher than your expected systolic pressure appears.
4. When the desired number is reached, release the **START** button to start measurement. Then continue to measure your blood pressure as described on the section "Normal Measurement".

Notes for Accurate Measurement

- Sit down in a comfortable position. Place the arm to be used for the measurement on a table or other support so that the center of the cuff will be at the same height as your heart.
- Relax for several moments before measurement.
- Remove constricting clothing and place cuff on bare arm.
- Unless your physician recommends otherwise, use left arm to measure pressure.
- Do not talk during measurement.

Measurement

- ❑ Do not measure immediately after physical exercise or a bath.
- ❑ Do not smoke or ingest caffeine at least 30 minutes prior to measurement.
- ❑ This device bases its measurements on the heartbeat. If you have a very weak or irregular heartbeat, the device may have difficulty determining your blood pressure.
- ❑ Should the device detect a condition that is abnormal, it will stop the measurement and display an error symbol. Refer to the section "Symbols" for the description of the symbols.
- ❑ This device is intended for use by adults only. Consult with your physician before using this device on a child. A child should not use this device unattended.
- ❑ Compare blood pressure taken at the same time each day.
- ❑ Do not cross legs and keep your feet flat on the floor during measurement.

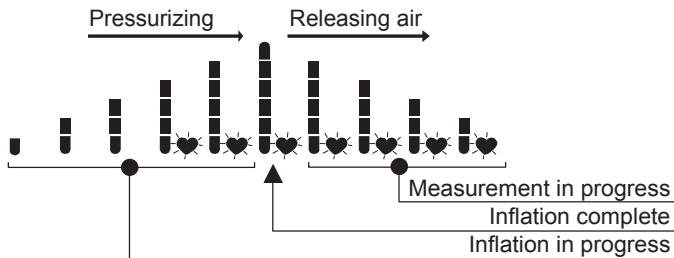
What is an Irregular Heartbeat

The UA-651BLE provides a blood pressure and pulse rate measurement even when an irregular heartbeat occurs. An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeats during the blood pressure measurement. It is important that you are relaxed, remain still and do not talk during measurements.


Note: We recommend contacting your physician if you see this symbol frequently.

Pressure Bar Indicator

The indicator monitors the progress of pressure during measurement

Troubleshooting

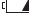Problem	Possible Reason	Recommended Action
Nothing appears in the display, even when the power is turned on.	Batteries are drained.	Replace all batteries with new ones.
	Battery terminals are not in the correct position.	Reinstall the batteries with negative and positive terminals matching those indicated on the battery compartment.
The cuff does not inflate.	Battery voltage is too low.  (LOW BATTERY mark) blinks. If the batteries are drained completely, the mark does not appear.	Replace all batteries with new ones.
The device does not measure. Readings are too high or too low.	The cuff is not applied properly.	Apply the cuff correctly.
	You moved your arm or body during measurement.	Make sure you remain very still and quiet during measurement.
	The cuff position is not correct.	Sit comfortably and still. Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.
	_____	If you have a very weak or irregular heartbeat, the device may have difficulty in determining your blood pressure.
Other	The value is different from that measured at a clinic or doctor's office.	At a clinic or doctor's office apprehension may cause an elevated reading. Home measurement reduces the effects of outside influences on blood pressure readings, supplementing the doctor's readings.
	_____	Remove the batteries. Place them back properly and try the measurement again.

Note: If the actions described above do not solve the problem, contact the dealer. Do not attempt to open or repair this product, as any attempt to do so will make your warranty invalid.

Maintenance

Do not open the device. It uses delicate electrical components and an intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact the authorized dealer in your area or our customer service department. A&D customer service can provide technical assistance and spare parts.

Technical Data

Type	UA-651BLE
Measurement method	Oscillometric measurement
Measurement range	Pressure: 20 - 280 mmHg Systolic pressure: 60 - 279 mmHg Diastolic pressure: 40 - 200 mmHg Pulse: 40 - 180 beats /min
Measurement accuracy	Pressure: ± 3 mmHg Pulse: ± 5 %
Power supply	4 x 1.5V batteries (R6P, LR6 or AA) or AC adapter (TB-233) (<i>Not included</i>)
Number of measurements	Approx. 700 times LR6 (<i>alkaline batteries</i>) Approx. 200 times R6P (<i>manganese batteries</i>) With pressure value 180 mmHg, room temperature 23 °C.
Classification	Internally powered ME equipment (<i>Supplied by batteries</i>)/Class II (<i>Supplied by adapter</i>) Continuous operation mode
Clinical test	According to ANSI/AAMI SP-10 1992
Wireless communication	VZ (MURATA Manufacturing Co.Ltd.) Bluetooth Ver. 4.0LE BLP

Technical Data

Operating conditions	+10 to +40 °C / 15 to 85 %RH / 800 to 1060 hPa
Transport/Storage conditions	-20 to +60 °C / 10 to 95 %RH / 700 to 1060 hPa
Dimensions	Approx. 96 [W] x 68 [H] x 130 [D] mm
Weight	Approx. 250 g, excluding the batteries
Ingress protection	Device: IP20
Applied part	Cuff Type BF 
Useful life	Device: 5 years (<i>when used six times a day</i>) Cuff: 2 years (<i>when used six times a day</i>)

Accessory AC adapter

Please contact your local A&D dealer for purchasing. The AC adapter is required to be inspected or replaced periodically.

TB-233	Input: 120V Output: 6V  500mA  139°C 2A
--------	--

Accessories are sold separately

Cuff

Part Number	Cuff Size	Arm Size
UA-420	AccuFit™ Plus Cuff	8.6"-16.5" (22-42cm)
UA-290	Medium Cuff	9.0"-14.6" (23-37cm)
UA-291	Large Cuff	12.2"-17.7" (31-45cm)

Arm size: The circumference at the biceps.

AC adapter: TB-233

Note: Specifications are subject to change without prior notice. This product is made in Japan.

Technical Data

FCC CAUTION

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment and meets the FCC radio frequency (RF) Exposure Guidelines in Supplement C to OET65. This equipment has very low levels of RF energy that it deemed to comply without maximum permissive exposure evaluation (MPE). But it is desirable that it should be installed and operated keeping the radiator at least 20 cm or more away from person's body (excluding extremities: hands, wrists, feet and ankles).

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Contact information

Our products are designed and manufactured using the latest scientific and technological methods, and offer accurate, easy to use, home monitoring and treatment options. Our full product line includes:

- Digital Blood Pressure Monitors
- Manual Blood Pressure Monitors
- Blood Pressure Cuffs
- Personal Health Scales
- Digital Thermometers
- Activity Monitors
- Wireless Products

Manufactured for:

A&D Engineering, Inc.
1756 Automation Parkway
San Jose, CA 95131 U.S.A.
www.andmedical.com
1-888-726-9966

Warranty

LIMITED WARRANTY

A&D Medical

If you have a Supply Agreement with A&D, the terms of your Supply Agreement exclusively govern your warranty and remedies, and this Limited Warranty does not apply to you.

For purchasers within the US only:

If you do not have a Supply Agreement with A&D and you are a CONSUMER (a purchaser for personal or household use), the Consumer Term of the Limited Warranty below applies to you.

If you do not have a Supply Agreement with A&D and you are NOT a Consumer, the Commercial Term of the Limited Warranty below applies to you.

[For purchasers outside the US, NO warranty is provided by A&D – please contact your local distributor or dealer.]

Product	Consumer Warranty Term	Commercial Warranty Term
UA-651BLE Monitor	5 Years	2 Years
Cuff	2 Years	1 Year

Limited Warranty:

A&D Medical (“A&D”) warrants to the first purchaser (“You”) that the A&D product You purchased (the “Product”) will be free from defects in material, workmanship and design for the applicable Warranty Term stated above from the date You purchased the Product under normal use. This Limited Warranty is personal to You and is not transferable. If the Product is defective, then (i) if You are a Consumer, You return the Product to the retailer You purchased it from (if within such retailer’s return time frame) or You return it to A&D in accordance with the procedure set forth below, or (ii) if You

Warranty

are NOT a Consumer, You return the Product to A&D in accordance with the procedure set forth below. A&D's warranty obligation is limited to the repair or replacement, at A&D's option, of the defective Product that has been returned by You within the warranty period. Such repair or replacement will be at no charge to You. The repaired or replacement Product is warranted hereunder for the longer of the remainder of the original warranty period or 90 days from the date of shipment of the repaired or replacement Product. If you return the Product for warranty service to A&D, You must return the Product, freight and insurance prepaid, within the warranty period to the address set forth below, together with satisfactory proof of the date of Your purchase (such as a sales receipt or statement of online warranty registration) and a description of the defect.

A&D Medical
Attn.: Warranty Department
1756 Automation Parkway
San Jose, CA 95131
1-888-726-9966

Please contact in the US: A&D Medical, at the numbers set forth above for the appropriate shipping costs.

This Limited Warranty does not cover, and A&D will not be liable for (i) any shipment damage, (ii) any damage or defect due to misuse, abuse, failure to use reasonable care, failure to follow written instructions enclosed with the Product, accident, subjecting the Product to any voltage other than the specified voltage, improper environmental conditions, or modification, alteration or repair by anyone other than A&D or persons authorized by A&D, or (iii) expendable or consumable components.

Warranty

This Limited Warranty is the only warranty PROVIDED BY A&D; there are no other express warranties. If A&D cannot reasonably repair or replace the Product, A&D will refund the amount You paid for the Product (not including taxes), less a reasonable charge for usage. To receive a refund you must have returned the Product and all associated materials to A&D. The above remedy of repair, replacement or refund is your only and exclusive remedy. IN NO EVENT SHALL A&D BE LIABLE FOR ANY DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS, LOST INFORMATION OR REPLACEMENT COSTS, ARISING OUT OF YOUR USE OF OR INABILITY TO USE THE PRODUCT, INCLUDING, WITHOUT LIMITATION, ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF A&D HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. Some states do not allow the exclusion of incidental or consequential damages, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

No distributor, dealer or other party is authorized to make any warranty on behalf of A&D or to modify this warranty, or to assume for A&D any liability with respect to its products.

Bluetooth® devices carry the *Bluetooth*® logo mark.

iPhone, iPad and iPod are trademarks of Apple Inc., registered in the U.S. and other countries.

Google and Google logo are registered trademarks of Google, Inc.

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Google y el logotipo de Google son marcas comerciales registradas de Google, Inc.
A&D Company, Limited