User Guide

Vivify Pathways  Home TM

Version 11

TABLE OF CONTENTS

INTRODUCTION	1
UNPACKING THE KIT	2
GETTING STARTED	3
Turning on the Tablet	3
Welcome to the Program	3
Accepting Terms and Conditions	4
Logging In	5
Completing Health Survey Questions	5
Viewing Education Videos	6
Understanding the Tablet	6
Requesting a Call	7
ENTERING VITALS	8
Weight	9
Oxygen Level and Pulse	9
Blood Pressure	9
Blood Sugar	9
PACKING THE KIT.....	10
FREQUENTLY ASKED QUESTIONS.....	11
Incorrect Readings	11
Powering on Tablet	11
Reset Password	11
Viewing Health Data	11
Wireless Connectivity	11

INTRODUCTION

Thank you for participating in the care management program. Please read the introduction to the program.

Unpacking the Kit

After you open the box, follow the instructions to turn on the tablet. Keep it plugged in. Do not turn it off until you complete the program and return the kit. **IMPORTANT:** Keep the kit box for returning the kit, including the foam and strips of packing tape. The box to keep looks like a small suitcase. Store this box in a safe place to keep it from being lost or damaged.

Logging In

Create a password that you will remember. Write this password down. Keep it in a safe place. You will enter it every time you log in.

Entering Health Survey Responses and Vitals

When asked, select answers to the questions that best describe your current health. Follow the instructions for how to enter your vitals. Your care management team will be looking at your results. They may contact you to talk about your results.

Check-In Schedule

The tablet may make a noise, alerting you to check your vitals. Your care management team will decide how often this happens. Take your vitals by noon, unless your care management team tells you differently.

Repacking and Returning Your Health Kit

Your care management team will call you when the program is complete. They will tell you how to return the kit. Read *Packing the Kit* for details about how to repack the health kit.

DO NOT USE THIS PROGRAM FOR EMERGENCY OR URGENT CARE. IN AN EMERGENCY, CALL 911 OR GO TO THE NEAREST EMERGENCY ROOM. The program and program nurses and other representatives cannot diagnose your problems or suggest treatment. This program is not a substitute for your doctor's care. Your health information is kept confidential in accordance with the law. Any information and any device provided through this program is for your information only. Participation in the program and use of the device is voluntary, and your ability to use the device may differ depending on the nature of your medical condition. The program and device do not provide real-time monitoring or act as an emergency alert system, and are not intended for use when time-critical care is required.


UNPACKING THE KIT

The health kit contains devices for you to use during the care management program to record your vitals. All devices are ready to use with the tablet.

HEALTH KIT CONTENTS

The devices in the box may vary based on your program. For most programs, the box will have the following:

- Tablet
- Tablet stand
- Adapter and power cord
- Weight scale
- Blood pressure monitor
- Pulse oximeter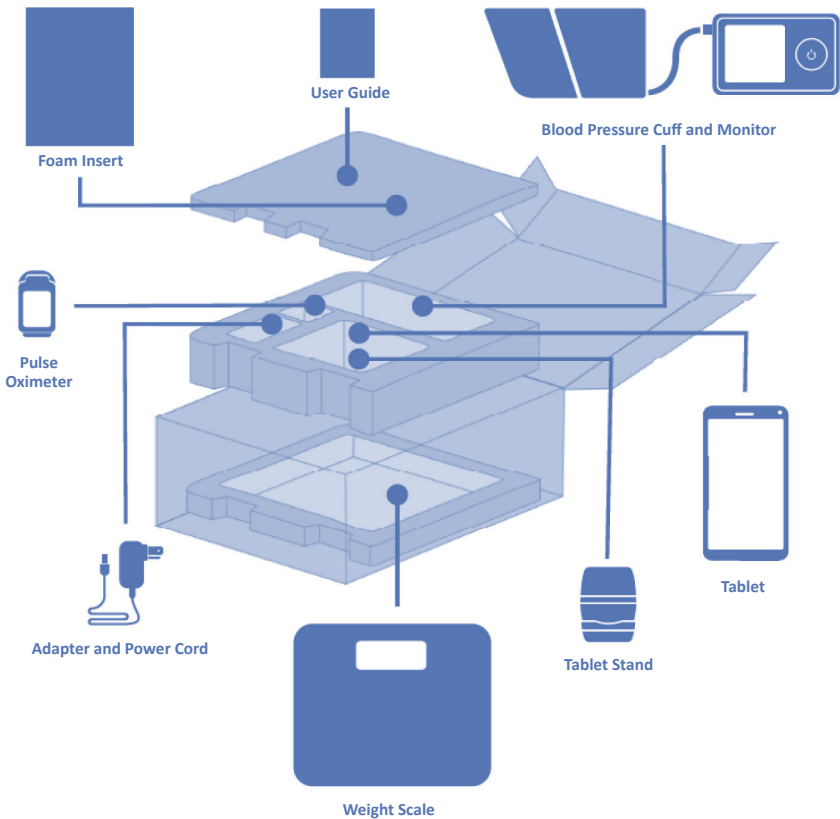

IMPORTANT: Keep the box that looks like a suitcase with foam inside for repacking. Store it in a safe area.

GETTING STARTED

TURNING ON THE TABLET

1. Unpack the kit.

Unpack the box on a table where you will use the tablet. You should be able to see the tablet when using the devices, so they can connect wirelessly.

2. Plug the power adapter into a nearby outlet. Insert the power cord into the tablet.

3. Turn on the tablet.

Press and hold down the power button for several seconds, then let go. Look at the image below. The power button is above the green dot on the tablet screen. Wait for the Welcome to the Program screen to appear.


4. Place the tablet in the tablet stand.

Keep the tablet on and plugged in until you complete the program and repack your kit.

WELCOME TO THE PROGRAM

When you select **OK** on the Welcome to the Program screen, the Date of Birth screen appears. In the **Date of Birth** box, enter your date of birth (MM/DD/YYYY) to confirm your identity. If the date of birth you enter does not match our records, you will see a message to contact support. After confirming your date of birth, the welcome (unpacking) video will appear.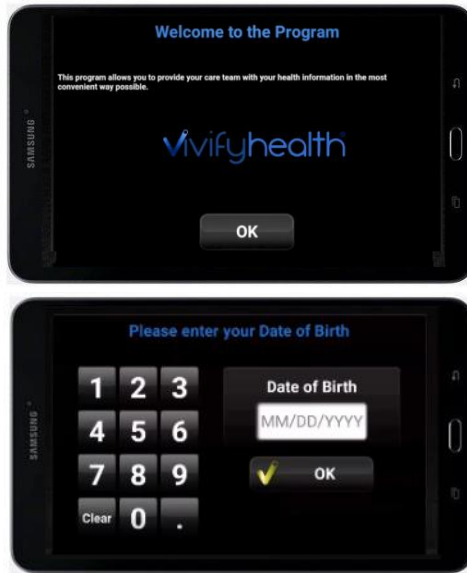

ACCEPTING TERMS AND CONDITIONS

After the video plays, read and select **I AGREE** to accept the End User License Agreement (EULA). *Note: You cannot continue until you agree to the program's terms and conditions.*

LOGGING IN

Create a password.

When the green login screen appears, create your password. Using the keypad, choose a four-digit number that you will remember. You cannot use the same number four times (1111). You cannot use numbers in order (1234). Write this number down. Store it in a safe place, so you can look at it later when logging in. Select **OK** to submit your password.

COMPLETING HEALTH SURVEY QUESTIONS

Questions appear in two ways:

Single Answer

When a question is asked, select an answer about your current health. Select **OK** to confirm. If you want to change your answer, select the correct answer. Then, select **OK** to confirm.

Multiple Answers


When a question is asked, select all answers about your current health. When you select one or more answers, **OK** will display. Select **OK** to confirm.

VIEWING EDUCATION VIDEOS

You may be asked to watch education videos or read about your health.

Select **Now** and then **OK** to watch the video now. Select **Later** and then **OK** to watch the video later.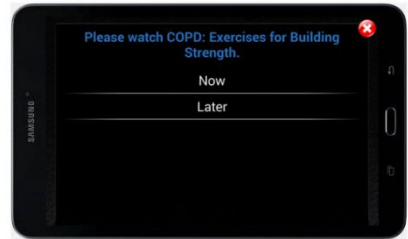

UNDERSTANDING THE TABLET

This is an image of the home screen when all tasks are done. The icons are based on your program. The icons show the tasks your care management team wants you to do. You may be asked to do these check-in tasks daily. You will be reminded by the colors changing and/or the tablet making a sound.


A green or orange checkmark means that you do not have any survey questions or vitals due. When a task is due, the icon color will change to gray, yellow, or red.

Gray	Yellow	Red	Green Check	Orange Check
Ready to start	Task is due now	Task is overdue	Task completed	Done for now

REQUESTING A CALL

You can ask for a call back from your care management team with your tablet. Call backs are not right away. Call backs happen during business hours. **This call back feature should not be used in an emergency.**

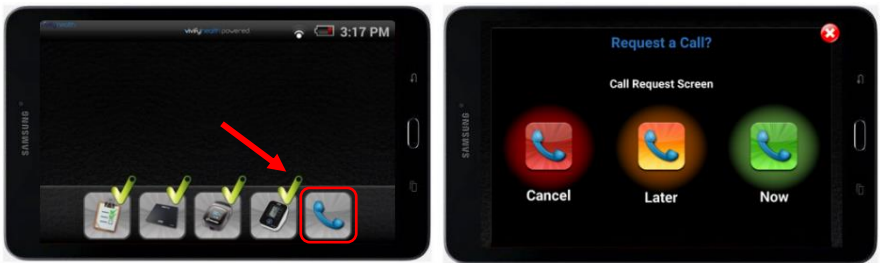To ask for a call back, select the phone icon in the bottom-right corner of the home screen. The Request a Call screen appears with up to three options and may include **Cancel**, **Later**, and **Now**.

Select **Now** to ask for a quick call back (usually within two hours). If you ask for a call within two hours of the close of business, you may get a call back the next business day.

Select **Later** to ask for a call back by the end of the next business day (usually within twenty-four hours). After you select Now or Later, the phone icon on the home screen will turn green. This shows that you are waiting for a call. The care management team will try to reach you by phone or by video call on the tablet.

Select **Cancel** if you change your mind or to exit the call request screen. The home screen appears. The phone icon is gray. The color change shows that you have canceled the call request.

IMPORTANT: Do not use the Request a Call feature in an emergency. If you need emergency care, call 911.

ENTERING VITALS

All the devices in this guide may not be in the kit or you may not need them all. The tablet icons will show you the devices to use. This is set by your care management team. Before you start, the devices need to be close to the tablet. When the device is not being used, it will turn off by itself. **Devices are only meant to be used in home for non-emergency use.**

Device Instructions

Before using the devices, read the device instructions. These instructions tell you how to use each device. The instructions include details about when to talk to your doctor before using each device. Any software for download in the instructions is not required for this care management program.

With the devices in your kit, do the following:

1. **Watch the Help video.** When the tablet asks you to record your data, select **HELP** if needed. A video will begin to show you how to use the device.
2. **Record your vitals.** All devices will send data to the tablet and record your data. As you use each device, wait a few seconds for the device to get your reading. A number will display on the device. Confirm that the reading on the tablet matches the reading on the device. Select **OK** to accept. Select **YES** to confirm.

Manual Entry

If a device does not record your data, select Manual Entry. In the boxes, record your data. Select **OK** to accept. Select **YES** to confirm.

If the number you enter is incorrect, select **NO** to start over and enter your data again. If you are using your own device, you must enter your data manually.

IMPORTANT: This product is not intended for active or continuous patient monitoring and not intended for high-acuity environments. If you need emergency care, call 911 immediately.

WEIGHT

The weight scale is for you to use during the care management program to record your weight. Weigh yourself in the morning, after you have gone to the bathroom and before you have had anything to eat or drink.

OXYGEN LEVEL AND PULSE

The pulse oximeter is for you to use during the care management program to record your oxygen level and pulse. Do not wear nail polish or fake nails while using the pulse oximeter. If your hands are cold, warm your hands.

BLOOD PRESSURE

The blood pressure monitor is for you to use during the care management program to record your blood pressure. Do not smoke, drink caffeinated beverages, or exercise within one hour of measuring your blood pressure. Also, if your doctor told you to take blood pressure medication, do it one hour before taking your blood pressure reading.

BLOOD GLUCOSE



The glucose meter is for you to use during the care management program to record your blood glucose. Store the test strips in their container and avoid exposure to moisture.

IMPORTANT: This product is not intended for active or continuous patient monitoring and not intended for high-acuity environments. If you need emergency care, call 911 immediately.

PACKING THE KIT

1. Get the health kit box from your safe place. The box looks like a small suitcase. Remove the user guide, packing tape, and the layers of foam.
2. Unplug the tablet from your wall outlet. Hold down the power button to turn off the tablet. Get all the devices that go in the health kit box.
3. Place the scale in the bottom of the health kit box. Put the middle foam tray on top of the scale. Wind up the tablet power cord.
4. Place the power cord and adapter, pulse oximeter, blood pressure monitor, tablet stand, and tablet in their slots.
5. Remove the tape from the health kit user guide. Place the last piece of foam on top of the tablet. Close and seal the box with the tape you just took out of the user guide. If you cannot find the tape, you can use packing tape instead.
6. Your care management team will tell you how to return the health kit.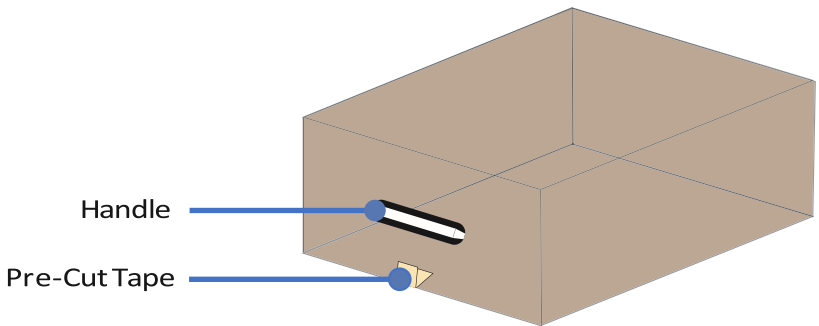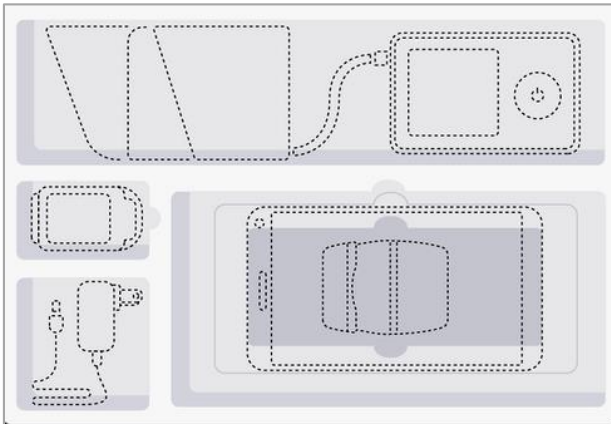

FREQUENTLY ASKED QUESTIONS

INCORRECT READINGS

Why does the blood pressure monitor show high readings or readings that are different from another blood pressure device I have used before?

Readings may vary from devices you already have. This is normal. Take your readings at the same time every day and while at rest. The most common cause for high readings is using the wrong sized blood pressure cuff. If the cuff does not fit or you have other questions about your health kit, please contact your care management team.

POWERING ON THE TABLET

The tablet is plugged in but will not power on.

Unplug the power cable from the tablet and press and hold down the power button for 10 seconds or until the screen shows “Samsung”. Once powered on, plug the power cable back into the tablet.

RESETTING YOUR PASSWORD

How do I reset my password?

If you forget your password, select “Help” on the login screen, then “request password reset” on the next screen. When your request is accepted by your care management team, the tablet will ask you to create a new password.

VIEWING HEALTH DATA

Can I view my data once it has been logged?

Once your vitals have been logged, you can look at the data with the Trend button. Not all tablets let you view your trend health data. Your care management team will review your information. If needed, they will contact you to talk about your trended results.

WIRELESS CONNECTIVITY

Why did the device not send my information to the tablet?

Keep the devices in your kit close to the tablet. If the tablet does not get your reading from a device, enter it on the tablet. It is important to do your daily readings. Your care management team will contact you to help if they see a lot of manual entries.

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